Verarbeitung & technische Daten CopraTemp Instructions for use & technical data CopraTemp

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general information: White Peaks CopraTemp Discs can be machined with any milling system. If required, data must be entered into the CAM module.

storage: Protect from heat and direct sunlight.

indication: White Peaks CopraTemp Discs are certified for intraoral use and therefor can be used in the patients mouth. The material is made for the dental interim solution. The temporary structure can stay for six months in the mouth of the patient.

finishing: For grinding use burs for use with acrylic. Polishing is possible with polishing paste and soft hair brushes. Pay attention not to overheat the material. For fixing the bridge use a temporary cement.

disposal: White Peaks CopraTemp Discs are made from pure PMMA. Waste can be recycled with other acrylics or can be disposed of through domestic residual waste system. PMMA is insoluble in water and is no hazard to groundwater.

rules: in occlusal region the minimum wall thickness has to be 1,0mm and 0,6 mm in cervical region; the connector profile has to be more than 10mm² in anterior region and more than 15mm² in posterior region; in posterior region max. two pontics are allowed between two abutments teeth